EN INSTRUCTION FOR USE

SoftFil® EasyGuide

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SoftFil® EasyGuide

ES MANUAL DE INSTRUCCIONES

SoftFil® EasyGuide

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SoftFil® EasyGuide

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SoftFil® EasyGuide

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SoftFil® EasyGuide

DESCRIPTION

SoftFil® EasyGuide is a sterile and single use injection kit containing :
- 1 stainless steel EasyGuide pre-hole needle in its protective cap
- 1 stainless steel blunt-tip needle (micro-cannula) graduated every centimeter along the tube in its protective cap.

SoftFil® micro-cannulas are available in various lengths and gauges.

INDICATIONS

The micro-cannulas are intended for hypodermal injection of body fluids or tissues, drugs or medical devices approved for human use.

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PRECAUTIONS

This device can be used only by or on the order of a physician and/or authorized healthcare professionals in accordance with local legislation and also trained in injection techniques. Before use, users shall inspect the injection site to make sure that there is no infection, inflammation, oedema, skin lesion, scar, birthmark, bony prominences or lipohypertrophy. The micro cannula is chosen based on two main parameters which determine how flexible the micro-cannula is:

- the length, according to the extent of the area to be treated.
 - the outside caliber (outside diameter), indicated in gauge (G) and in millimeters (mm), to be adapted to the viscosity of the product and to the injection plane.
- The technique, method and products used depend on experience and are left to the physician and/or authorized healthcare professional's discretion in order to ensure the best clinical results.

In case of significant bleeding at entry point, it is recommended to make a new entry point.

Users should perform under appropriate aseptic conditions and handle needles with precaution to avoid inadvertent sharps injury and getting infectious disease.

Micro-cannulas are fragile devices, to handle with caution. Any deflection, crushing or damage of the tube may lead to its breaking during its use. Avoid any sudden movement or force in case of resistance.

In case of breaking during the use, the patient should be addressed as soon as possible to an authorized surgeon who will take the appropriate measures. Do not reuse if the product or the pouch appears to be damaged.

For single patient use only - Contents supplied sterile

Do not reuse, reprocess, or resterilize. Reusing, reprocessing or resterilizing may compromise the structural integrity of the device and or lead to device failure, which in turn may result in injury or illness. Reusing, reprocessing or resterilizing may also create a risk of contamination of the device and/or cause infection or cross infection including, but not limited to, the transmission of infectious disease(s).

Any device which has been used or partially used must be destroyed in accordance with the local regulations concerning the disposal of hospital waste.

USE

- Inspect the packaging to confirm its integrity before opening.
- Prepare the skin area to be injected: make-up removal, cleaning, rinsing, drying, antiseptic to prevent a biofilm.
- Open the packaging.
- To fix the SoftFil® micro-cannula on the syringe, hold the micro-cannula by its plastic cover firmly and:
 - in case of syringes with fixed female thread: screw it towards the syringe tip.
 - in case of syringes with mobile female luer lock: push it strongly towards the syringe tip while twisting it clockwise. For optimal screwing, use a clamp if necessary.

- Create pre-hole at the identified entry point, with the SoftFil® EasyGuide pre-hole needle included in the sterile injection kit (Diagram A).
- Through this entry hole, insert the micro-cannula previously securely connected and locked to the syringe with one hand by sliding the micro-cannula along the SoftFil® EasyGuide pre-hole needle into the desired injection level (Diagram B). Then, remove the pre-hole needle from the hole and gently proceed with the injection (Diagram C) applying the least amount of pressure. Rely on the mark located on the hub indicating the orientation of the side port and the graduations every centimeter along the tube for determining the depth of the injection to favor a homogeneous treatment.

- To remove the SoftFil® micro-cannula from the syringe, hold the syringe and turn the pre-hole female luer lock counterclockwise firmly, then pull the micro-cannula backwards gently.
- Dispose off micro-cannulas and pre-hole needles in proper sharps container.

SIDE EFFECTS
Few complications as pain, redness, bruising, hematoma, swelling or oedema, may result from the injection technique itself.

Furthermore, repeated injections can cause tissue scarring.

VALIDITY / STORAGE CONDITIONS
Do not use after the expiry date.

The product must be stored in its original packaging, in a dry place and in the shelter of light.

SYMBOLS USED ON PACKAGING
REF Product reference / **U** Use until / **LOT** Batch number / Date of manufacture / **D** Do not reuse / **R** Do not resterilize / **D** Do not use if package is opened or damaged / **STERILE** Sterile product. Method of sterilization – Ethylene oxide / **Ø** Dimension / **I** See the Instructions for use leaflet / **M** Manufacturer / **K** Keep away from moisture / **S** Keep away from sunlight / **MC** Micro-cannula / **PH** Pre-hole needle / **L** Latex free / **R** Only Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician / **CE** CE Mark obtained in compliance with Directive 93/42/EEC related to medical devices. 0459 is the number of the Notified Body. CE Mark obtained in 2016. **MD** This product is a medical device **○** Single sterile barrier system **UDI** Unique Device Identifier

COLOUR CODES ON HUBS AND LABELS
Black = 22G / Dark blue = 23G / Orange = 25G / Medium grey = 27G

Any serious incident related to the use of this product must be reported to the device manufacturer and the competent authority of the country/state in which the user or patient is established.

All returned products must first be decontaminated, and accompanied by a decontamination certificate. If no certificate is provided, the claim will be rejected.

SoftFil® is a registered trademark of Soft Medical Aesthetics
Soft Medical Aesthetics is EN ISO 13485-certified
SoftFil® EasyGuide, international patent pending

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